



The Temp That Will Earn Its Way Into Your Lineup

Radica is a revolutionary, lab-processed provisional material that combines exceptionally high flexural strength characteristics with best-in-class esthetics. This unusual combination of brawn and beauty enables you to insert Radica temps into your lineup in a variety of roles. From try-in application to long-term provisional appliance – Radica gives you options you simply won't find with other resins.

You will quickly discover that Radica is easy to use and work with for all relines, repair and placement procedures. What's more, Radica's monomer-free formulation and extraoral cure means lower incidences of sensitivity—it's a tissue friendly and biocompatible system. Radica's superior strength and wear resistance will give you peace of mind when addressing full mouth reconstruction and implant cases.

In the unusual event of a crack or break, repairs are easy with Radica. First roughen the fracture surface with medium burs, and then clean thoroughly. Treat with Prime and Bond® NT™ Universal Adhesive and cure (making sure to avoid applying adhesive to tooth surfaces). Then simply apply TPH® 3 MicroMatrix Restorative as repair material and light cure.



Expand Your Expectations

With Radica, you can create lifelike diagnostic restorations that will help manage patient expectations. Unlike other try-in applications, it can also serve as a high-quality provisional treatment guide.



Current wax diagnostic option



New esthetic diagnostic try-in available with Radica


Image courtesy of Dr. Thanos Kristallis

Success in 8 Simple Steps


- 1 Test fit the restoration onto the finished tooth preparations. Evaluate and adjust for complete seating with acceptable occlusion.
- 2 Clean the internal surface with aluminum oxide or lightly grind with ¼ round bur.
- 3 Reline restoration with PMMA acrylic or a material such as Integrity™ Temporary Crown and Bridge Material. Consult manufacturer's instructions for proper reline technique.
- 4 Reseat the case and allow excess material to express.
- 5 Remove excess material.
- 6 Upon removal of the restoration, finish and polish margins.
- 7 Apply a temporary cement and seat restoration.
- 8 Finish the case and polish with a product such as Enhance® Finishing System or PoGo® One Step Diamond Micro-Polisher.

SCIENTIFIC STUDIES CONFIRM RADICA PERFORMANCE EXCEEDS ALL OTHER TEMPORARY RESINS


ISO 10477 Flexural Strength Comparisons of Radica with other Materials

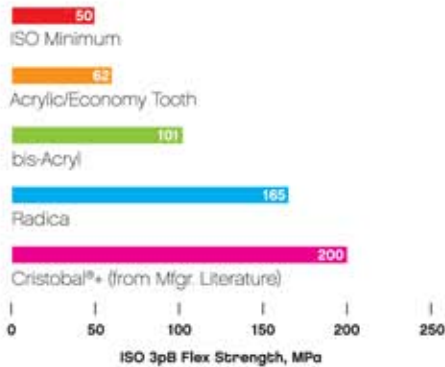

Radica Shows Very Low Wear Against Antagonists

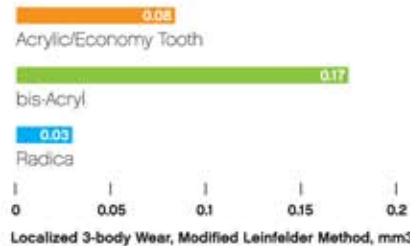

Long-Term Durability of Radica Restorations

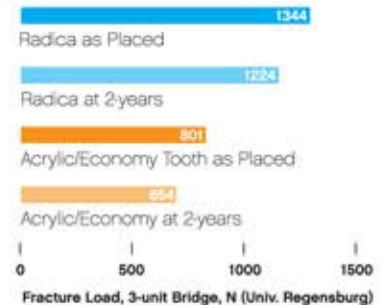

With Radica Provisional & Diagnostic Resin, you will have a material that...

- Provides a lifelike tooth-colored diagnostic for case presentation
- Offers the optimum in a low-wear, highly-durable material for long-term provisional restorations
- Delivers premium, high-end esthetics to your patients as they transition to their fixed restoration
- Yields excellent strength for confidence in performance

DENTSPLY
PROSTHETICS

The chemistry of a great smile.™

radica™
provisional & diagnostic resin

DENTSPLY International
DENTSPLY Ceramco
570 West College Avenue
York, PA 17404
1.800.487.0100 USA
www.ceramco.com

Distributed by:
DENTSPLY Canada
161 Vinyl Court
Woodbridge, Ontario L4L4A3
1.800.263.1437 Canada

LabLocator 
www.ceramco.com/radica

Ronly
Patent Pending
PC#090821
Rev. 10/07