

### PLACEMENT TECHNIQUE



1 Test fit of the restoration onto finished tooth preparations. Evaluate and adjust for complete seating with acceptable occlusion.



2 Clean the internal surface with aluminum oxide or lightly grind with 1/4 round bur.



3 Reline and replace restoration with an MMA acrylic such as Biolon®, a bis-GMA such as Integrity™ Temporary C&B Material or Temporary Bridge Resin from Caulk. Consult manufacturer's instructions for proper relining technique.



4 Reseat the case and allow excess material to express.



5 Remove excess material.



6 Upon removal of the restoration, finish and polish margins.



7 Apply a temporary cement and seat restoration.



8 Finish the case and polish with a product such as Enhance® Finishing System or PoGo® One Step Diamond Micro-Polisher.

# radica™

provisional & diagnostic resin

## ONE TOUGH TEMP



Image courtesy of Dr. Thanos Kristallis

Radica™ provides enhanced strength, durability, and wear resistance, with an improvement in esthetics over current direct provisionals. And while Radica is new, it's already proven itself to be one tough customer. Radica has been validated by field-testing in multiple laboratories, with over 90% of evaluators expressing a preference for Radica over their current processes. If you're looking for a temp that won't crack under pressure, or break your bottom line, look no further than Radica. It's one tough temp.

With Radica Provisional & Diagnostic Resin, you will have a material that...

- Provides a life-like, tooth-colored diagnostic for case presentation.
- Offers the optimum in a low-wear, highly-durable material for long-term provisional restorations.
- Delivers premium, high-end esthetics to your patients as they transition to their fixed restoration.
- Yields excellent strength for confidence in performance.

DENTSPLY International Inc.  
570 West College Avenue  
York, PA 17405  
1.800.487.0100  
www.ceramco.com www.dentsply.com  
©2007 DENTSPLY International Inc. All rights reserved.

Distributed by:  
DENTSPLY Canada  
161 Vinyl Court  
Woodbridge Ontario L4L4A3

LabLocator  
www.ceramco.com/radica  
Rx only CE 0086  
Potent Pending  
PN# 908034 Rev. 08/07

## REPAIR TECHNIQUE



Roughen fracture surface with medium burs; clean thoroughly.



Treat with Prime and Bond® NT™ Universal Adhesive, and light cure.

Avoid applying adhesive to tooth surfaces.

Apply TPH® 3 MicroMatrix Restorative as repair material, and light cure.